

Disposable Dressing Set

一： Product name, model and specification

Product name: disposable dressing Set

Model specifications: Cleaning type, Debridement type, PICC(central venous catheter) type

二： Name, address, production address, contact information and after-sales service unit of the registered person/manufacturer

1. Name: Hunan Zhenheyikang Medical Instrument Co., LTD

2: Address: NO.6 Building Jingxiang Energy, No.55 Xianguang east road,Gaoxin district Xiangtan, Hunan

3:Production address:NO.6 Building, NO.8 Building,3rd and 4th floor Jingxiang Energy, No.55 Xianguang east road,Gaoxin district Xiangtan, Hunan,NO 19 Chutian Road, Yuetang District, Xiangtan City, Hunan

4:Contact number:+86 17711627084

5:After-sales service unit:Hunan Zhenheyikang Medical Instrument Co., LTD

三、 Manufacturing License NO : Hunan Medical Products Administration Production Licence No.20150013

四、 Product Registration Certificate No、 Product Technical Requirements No: HNFDA Certified 20182140083

五、 Structure and composition, scope of application

Cleaning type: tray, plastic tweezers, medical gauze block, iodophor cotton ball, alcohol cotton ball, towel, disinfection forceps; Debridement type: surgical blade, medical suture needle, non-absorbable surgical suture, tray, plastic tweezers, medical gauze block, iodine cotton ball, alcohol cotton ball, towel, disinfection forceps; PICC(central venous catheter) type: blue wrapping paper, disinfection cotton swab, disposable medical alcohol disinfection tablet, disposable sterile rubber surgical gloves, pull adhesive tape, self-adhesive transparent dressing, non-woven gasket composition. The accessories in the package should be pollution-free, no damage, no odor, tasteless phenomenon. Plastic accessories should be smooth, uniform color, without burrs, flash and cracks. Product aseptic supply, ethylene oxide sterilization, ethylene oxide residue should be $\leq 10\mu\text{g/g}$.

For clinical patients in medical institutions to use for dressing change surgery, iodophor cotton ball, alcohol cotton ball for hand, skin and mucous membrane disinfection.

六： Use method: open the bag can be used.

七： Contraindications, precautions, warnings and reminders

Product contraindications: None

Damaged packaging prohibited use, expired products prohibited use;

Disposable use, please destroy after use;

Avoid high temperature, humidity, direct sunlight;

八： Interpretation of graphics, symbols, abbreviations and other contents used in medical device labels




一次性使用



参考使用说明书



包装破损切勿使用

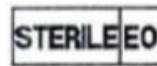

环氧乙烷灭菌



无菌

九. Installation and use instructions or illustrations

This product does not need to be installed, open the bag can be used.

十, Product maintenance and maintenance methods, special storage, transportation conditions, methods

Products should be stored in relative humidity $\leq 80\%$, non-corrosive gas and well-ventilated indoor, ordinary transport, in the transportation, storage process should be kept clean and pollution-free, strictly prohibited rain and snow immersion.

十一： Date of production, shelf life or expiration date

Valid for two years, production batch number, production date, expiry date please refer to the small package or qualification label.

十二, Accessories list, including accessories, appendix, lossitage replacement cycle and replacement method description

This product has no accessories

十三. Date and version of preparation or revision of the specification

This specification was revised in February 2023, version 2023.