

# Medical Protective Mask ( Head-mounted )

## 一、 Product name, model and specification

Product name: Medical Protective Mask

Model: folding type (non-sterile)、folding type (sterile); Specification: L 16cm×10cm、 16cm×9cm、 S 码 14cm×9cm

Model: arch type (non-sterile)、 arch type (sterile); specification: L 14cm×11cm、 M 13cm×13cm、 S 13cm×12cm

## 二、 Name, address, production address, contact information and after-sales service unit of the registered person/manufacturer

- 1、 Name: Hunan Zhenheyikang Medical Instrument Co.,LTD
- 2、 Address: No.6 Building Jingxiang Energy, No.55 Xiaguang east road, Gaoxin district Xiangtan, Hunan
- 3、 Production address: NO.6 Building, NO.8 building, 3<sup>rd</sup> and 4<sup>th</sup> floor Jingxiang Energy, No.55 Xiaguang east road, Gaoxin district Xiangtan, Hunan, NO 19 Chutian Road, Yuetang District, Xiangtan City, Hunan
- 4、 Contact number: +86 17711627084
- 5、 After-sales service unit: Hunan Zhenheyikang Medical Instrument Co.,LTD

三、 **Manufacturing License NO:** Hunan Medical Products Administration Production Licence No. 20150013

四、 **Product Registration Certificate No, Product Technical Requirements No:** HNFDA Certified 20202140323

## 五、 Structure and composition, scope of application

It is composed of mask body, nose clip and mask belt. The inner body and outer layers of the mask are ordinary non-woven cloth, and the middle layer is melt-blown cloth. The nose clips are made of polypropylene material, and the mask bands are made of elastic bands. Non-sterile supply.

Suitable for wearer hygiene care in a general medical setting without body fluids and splash risk.

六、 **Use method:** open the bag can be used

## 七、 Contraindications, precautions, warnings and reminders

Product contraindications: None

Damaged packaging prohibited use, expired products prohibited use;

Disposable use, please destroy after use

Avoid high temperature, humidity, direct sunlight

## 八、 Interpretation of graphics, symbols, abbreviations and other contents used in medical device labels



一次性使用



参考使用说明书



包装破损切勿使用



未灭菌

## 九、 Installation and use instructions or illustration

It is recommended to use no more than 4 hours.

Instructions for use: 1. Open the package; 2 hold the protective mask with one hand and the side with the nose clip is turned outward; 3. Cover the protective mask over the nose, mouth and chin to the face; 4. pull the lower strap over the head with the other hand and the ears behind the neck; 5. then pull the upper strap to the middle of the head; 6. place the fingertips on the metal nose clip, start from the middle position, press the nose clip with fingers inward, and move to both sides respectively, and shape the nose clip according to the shape of the nose bridge. Considering the face difference, the head band is equipped with an adjustable buckle. Adjust and wear, refer to the following figure.



## 十、 Product maintenance and maintenance methods, special storage, transportation conditions, methods

Products should be stored in relative humidity  $\leq 80\%$ , non-corrosive gas and well-ventilated indoor, ordinary transport, in the transportation, storage process should be kept clean and pollution-free, strictly prohibited rain and snow immersion.

**十一、 Date of production, shelf life or expiration date**

Valid for three years, production batch number, production date, expiry date please refer to the small package or qualification label.

**十二、 Accessories list, including accessories, appendix, lossitage replacement cycle and replacement method description**

This product has no accessories

**十三、 Date and version of preparation or revision of the specification**

This specification was revised in February 2023, version 2023.